

Kii® Dissecting Balloon Access Systems

SETUP GUIDE




1 Insert introducer until introducer and seal are locked into place.



2 Insert dissecting balloon into incision site with inflation port perpendicular to the abdomen.



3 Remove introducer.

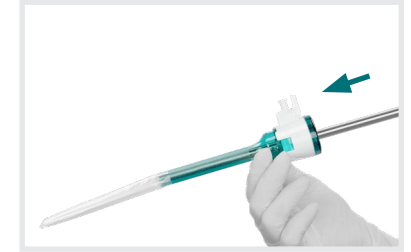

4 Insert 10mm endoscope.



5 Connect the inflation bulb and twist the Luer lock connector clockwise onto the stopcock until firmly attached.



6 Inflate the dissecting balloon under visualization by compressing the inflation bulb.



7 After completing dissection, untwist the Luer lock connector counterclockwise and disconnect the inflation bulb.



8 Remove the endoscope for complete balloon deflation.

Visit www.appliedmedical.com/DissectingBalloons for more information

This information is not intended to replace the Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.