


Kii™ Structural Balloon Access System

SETUP GUIDE

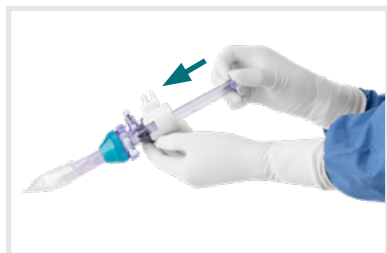


- 1** Insert the obturator until the obturator and seal lock into place.




- 2** Insert the structural balloon access system into the incision with the balloon inflation port facing anteriorly.



- 3** Connect the clear end of the inflation bulb to the balloon inflation port and inflate the structural balloon by squeezing the bulb three to four times.

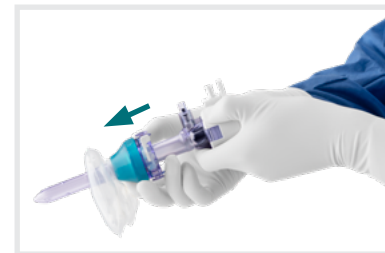

- 4** Slide the gel cone into the incision and fixate it by wrapping sutures around the suture ties or by locking the lever.

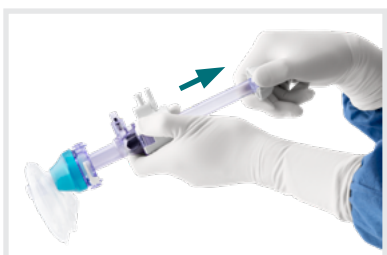

- 5** Remove the obturator.



- 6** Connect the insufflation tubing to the stopcock valve to insufflate the space.



- 7** To prepare the device for removal at the end of the procedure, connect the black end of the inflation bulb to the balloon inflation port and deflate the structural balloon by squeezing the bulb.

Please contact your Applied Medical representative for more information on availability. This information is intended for dissemination exclusively to healthcare professionals and is not intended to replace labeling and Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.

© 2025 Applied Medical Resources Corporation, Rancho Santa Margarita, CA. All rights reserved.

Applied Medical, the Applied Medical logo design, and marks designated with the symbol ™ are trademarks of Applied Medical Resources Corporation, registered in one or more of the following countries: Australia, Canada, Japan, South Korea, the United States, the United Kingdom, and/or the European Union.

04/2025

669445-EN-A